STERANIOS 2%

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An Ecolab Company

5L

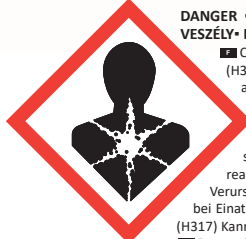
4K0382986005LEMR-220621
3112070F/21JUN22/S0

STERANIOS 2%

UFI : 4V7W-7KGH-JF07-EG0N

DANGER • GEFAHR • GEVAAR • PELIGRO • PERIGO • ΚΙΝΔΥΝΟΣ • ΟΠΑΧΟ • VESZÉLY • NIEBEZPIECZEŃSTWO • NEBEZPEČÍ • NEBEZPEČENSTVO • TEHLÍKE

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GB. High-level disinfectant

INDICATIONS: High-level disinfectant for thermosensitive instruments and endoscopy equipment. **COMPOSITION:** Glutaraldehyde solution (2%) buffered to pH 6, surfactant, colorant and excipients. **MICROBIOLOGICAL PROPERTIES:** Bactericidal* according to EN 13727/ EN 14561* in 5 min. Yeastcidal* according to EN 13624/EN 14562* in 5 min. Fungicidal* according to EN 13624/EN 14562* in 10 min. Tuberculocidal* according to EN 14348/EN 14563* in 10 min. Mycobactericidal* according to EN 14348/EN 14563* in 10 min. Virucidal* according to EN 14476/EN 17111* in 10 min. Sporicide against C. Difficile* according to EN 17126* in 60 min. *clean conditions. **INSTRUCTIONS FOR USE:** Ready-to-use solution. Do not dilute. Carefully clean and rinse the medical device before soaking it in STERANIOS 2%. **Pre-disinfection step:** Clean the medical device with a detergent or pre-disinfectant product. Rinse thoroughly. **Disinfection step:** Pour the solution. Respect the indicated contact time for the required antimicrobial activity. Keep the soaking bath covered during the contact time. Do not add any product into STERANIOS 2%. Rinse thoroughly with sterile or filtered water (0.2 µm) to eliminate all traces of the product. For endoscopic equipment: the internal and external parts of the medical device. Dry with a single-use towel. Keep the medical device as aseptically as possible up to use it again. To guarantee the efficacy of the disinfectant, check the glutaraldehyde level with a STERANIOS 2% testing strip at the start of the session, when restarting after an interruption and at least once a day. Maximum storage of the immersion bath: 30 days except 7 days for France (Instruction no. 220 of 4 July 2016). Scientific dossier (efficacy, compatibility, etc.) available on request. **PRECAUTIONS FOR USE:** (P261) Avoid breathing vapours. (P280) Wear protective gloves/ eye protection/ face protection. (P284) In case of inadequate ventilation wear respiratory protection. (P304 + P340) IF INHALED: Remove person to fresh air and keep comfortable for breathing. (P342 + P311) If experiencing respiratory symptoms: Call a POISON CENTER/doctor. (P501) Dispose of unused product and its container as hazardous waste. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State. Medical Device. Reserved for professional use. Made in France.

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